



COVID-19 Antigen Nasal Test Kit – Self Test Instruction Guide

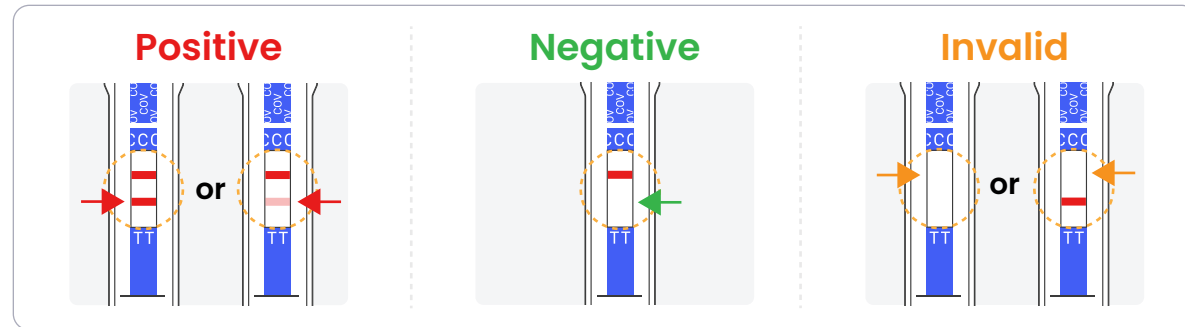


To watch a demonstration video, visit <https://www.pathdx.com.au/nasalpenhometest>. Freecall us on 1800 728 439 (9am to 7pm AEST).

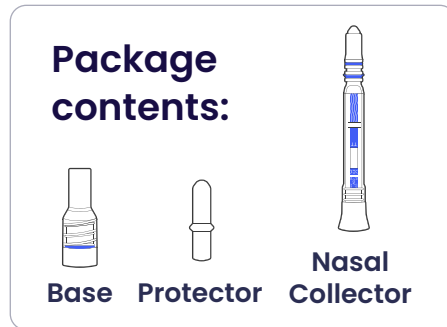


Note: Use test only one time. Test within first 7 days of symptoms. Testing by adult or under adult supervision only.



Wash your hands before the test.

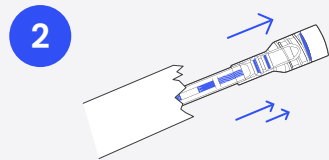






Remove the test from the packaging.

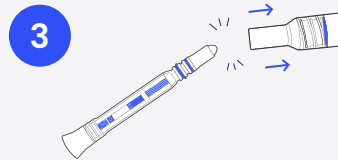

Take the nasal collector out of the base.



Remove the protector from tip.

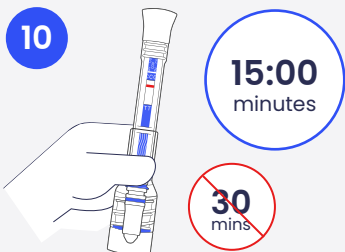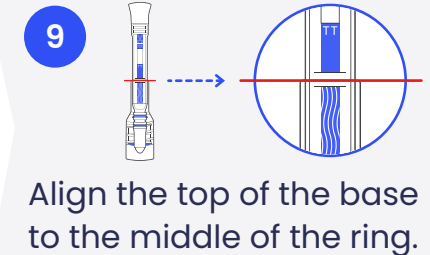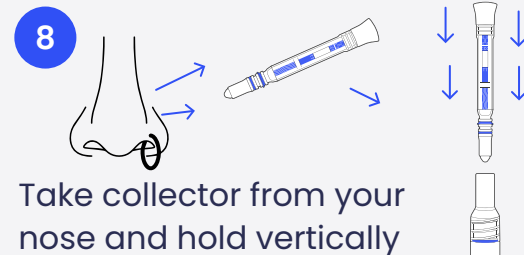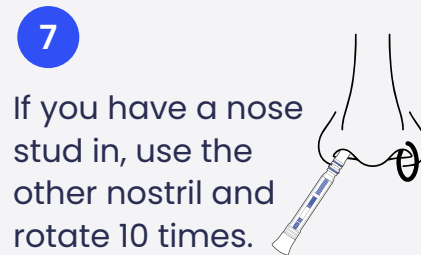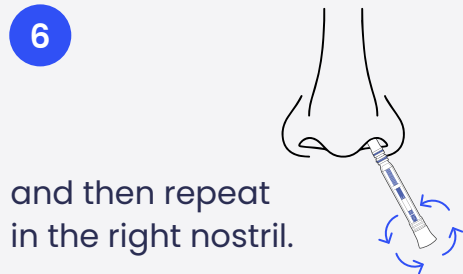

Wait 15 minutes, then check your results. **Results will be invalid after 30 minutes.**

